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SK

# CONTENTS

Safety precautions	31
Introduction	32
Precautions before use	33
Functions	33
Usage	34
Practical considerations when taking a temperature	34
How to take a temperature	35
Important information when taking a temperature	35
Three colors backlit LCD display	35
Description	36
Setting and function	37
Instructions for calibration	39
Changing the batteries	39
Technical characteristics	40
Troubleshooting	41
Warranty	42

## SAFETY PRECAUTIONS

- Follow the maintenance advice stipulated in this instruction manual.
- This device may be used for professional purposes or for personal home use.
- This device must only be used in an ambient temperature range of between 10°C and 40°C.
- This device must always be kept in a clean, dry area.
- Do not expose this thermometer to electric shocks, water or other liquids.
- Do not expose this thermometer to extreme temperature conditions of > 55°C or < -20°C.
- Do not use this device in relative humidity higher than 85%.
- The protective glass over the lens is the most fragile part of the thermometer.
- Do not touch the glass of the infrared lens with your fingers.
- Clean the glass with a cotton bud lightly moistened with 95% alcohol.
- Do not expose the thermometer to sunlight.
- Never drop the device.
- Do not attempt to repair this device yourself. If you have any problem, please, contact authorised service department.

EN

The manufacturer reserves the right to alter the specification of the product without prior notification.

## INTRODUCTION

The non-contact infrared thermometer Helpmation has been developed by using the latest infrared technology. This technology allows temporal artery (TA) temperature to be taken at a distance of max. 3-5 cm away from the forehead. It has been demonstrated that this method of TA temperature measurement is more precise than the tympanic thermometry and better tolerated than rectal thermometry (1). However, as with other types of thermometer, it is essential to use the thermometer properly in order to obtain reliable and stable results.

(1) „Greenes D, Fleischer G. Accuracy of a Noninvasive Temporal Artery Thermometer for Use in Infants. Arch Pediatr Adolesc Med 2001, 155:376.“

### NORMAL TEMPERATURES ACCORDING TO AGE

Age	°C	°F
0-2 years	36.4-38.0	97.5-100.4
3-10 years	36.1-37.8	97.0-100.0
11-65 years	35.9-37.6	96.6-99.7
> 65 years	35.8-37.5	96.4-99.5

## PRECAUTIONS BEFORE USE

The thermometer Helpmation is pre-set at the factory. It is not necessary to calibrate the device when starting.

In order to obtain reliable and stable results, you are advised each time there is a significant change in the ambient temperature due to a change in environment, to allow the thermometer to acclimatize to this ambient temperature for 15 to 20 minutes before using it. It is important to allow 1 minute interval between two measurements.

## FUNCTIONS

1. Special design to take the Human Body Temperature from a distance.
2. Reliable and stable measurement, thanks to the advantage Infrared Detection System.
3. Audible alarm if temperature is too high.
4. Three colors backlit LCD display.
5. Temperature can be displayed in either Celsius or Fahrenheit.
6. Automatic power-off (30 seconds) to save energy.
7. Practical, easy to use.
8. Longevity use (100.000 reading).

The thermometer can also be used to measure the temperature of a baby-bottle or bath, or room temperature (by using the Surface Temp or Room function). This function is in accordance with the Directive 89/339/EEC Electromagnetic compatibility.

## **USAGE**

1. Install batteries (please see Changing the batteries)
2. For the first use or when inserting new batteries wait around 10 minutes for the warm-up of the device.
3. Aim towards the forehead, from a distance of max. 3-5 cm, press the measuring button SCAN, the temperature is displayed in 1 second.
4. Before taking the temperature, make sure to remove hair and perspiration from the forehead.

## **PRACTICAL CONSIDERATIONS WHEN TAKING A TEMPERATURE**

In order to ensure that precise and accurate temperature measurements are obtained, it is essential that each user has received adequate information. Temperature should be taken in a neutral context. The patient must not have undertaken vigorous physical activity prior to taking the temperature.

Be aware of physiological variations in temperature which must be taken into consideration when evaluating the results:

- temperature increases by 0,5°C between 6am and 3 pm
- women have a temperature that is higher, on average, by around 0,2°C
- women temperature also varies in accordance with their ovarian cycle. It rises by 0,5 °C in the second half of the cycle and at the early stages of pregnancy.
- when sitting, temperature is lower by about 0,3 - 0,4°C than when standing

## HOW TO TAKE A TEMPERATURE

Aim at the forehead, over the right temporal region, from a distance of max. 3-5 cm, press the button SCAN and the temperature is instantly displayed.

Do not measure the temperature on other parts of the body (hand, abdomen, etc.). The measured results will not be correct.

## IMPORTANT INFORMATION WHEN TAKING A TEMPERATURE

Please observe the following before any temperature measurement to ensure a stable and reliable result:

- push back hair from the forehead
- wipe away any perspiration and make-up from the forehead
- avoid any drafts (e.g. from nasal specs, air conditioning ...)
- allow a 1 minute interval between two measurements
- each time there is a significant change in the ambient temperature due to a change in environment, to allow the thermometer to acclimatise to this ambient temperature for at least 15 minutes before using it

## THREE COLORS BACKLIT LCD DISPLAY

Green: normal temperature  $\leq 37.3^{\circ}\text{C}$  ( $99.1^{\circ}\text{F}$ )

Orange color: increased temperature  $37.4$  to  $37.9^{\circ}\text{C}$  ( $99.2$  to  $100.2^{\circ}\text{F}$ )

Red: high fever  $\geq 38^{\circ}\text{C}$  ( $100.3^{\circ}\text{F}$ )

# DESCRIPTION HELPMATION THERMOMETER

# SETTINGS

## Switch ON

If the thermometer is off, press the SCAN button once and the thermometer will turn on (the LCD display will light up and the last measured value will appear on the display).

Press the SCAN button again to measure the desired temperature.

## Temperature measurement

- **BODY temperature** - BODY must be shown on the display. Now hold the thermometer max. 3-5 cm in front of the center of the forehead and press the SCAN button. The display shows the measured temperature.
- **SURFACE temperature** - SURFACE must be shown on the display. Now press the SCAN button. The display shows the measured temperature.
- **ROOM temperature** - the display must show ROOM. Now press the SCAN button. The display shows the measured temperature.
- **Note:** The factory setting is in BODY mode

## Change measurement settings

Press the MODE button to change the individual BODY, SURFACE and ROOM measurement modes.

## Alarm

In BODY mode, if the temperature is lower than the set value (default value is 38 °C). The alarm beeps once. If the temperature is higher than or equal to the alarm temperature (default value is 38 °C), the alarm beeps „DI“ 5 times. If the temperature in SURFACE mode is higher than 60 °C, HI will appear on the display.

## Selection of measurements in °C or °F

Turn on the thermometer, then press the °C/°F button and select the desired measurement method in °C or °F.

## ALARM ON / OFF

Turn on the thermometer, then press and hold both MODE + °C/°F buttons for 3 seconds. The display shows „ON“ - the alarm is on, OFF - the alarm is off. Press the °C/°F button to toggle between ON and OFF and press the MODE button to confirm the alarm on or off selection. To exit the setting, press the SCAN button.

### NOTICE!

**To obtain the internal temperature of the body, always use the „BODY“ mode. Please, make sure to select „Sufrace temp“ mode for an external area reading.**

## CALIBRATION OF THE THERMOMETER

Thermometer factory-set. Depending on various skin types and thickness, there may be temperature differences.


Turn on the thermometer, then press and hold the MODE button for 3 seconds. The display shows the information „F4“. Press the °C/°F button to change the setting by 0.1 °C. Save the setting by pressing the MODE button. To exit the setting, press the SCAN button. The factory setting of the thermometer is 0.0 °C. The thermometer can be readjusted between -1.0 °C and 1.0 °C.

In the cases of seasonal or environmental changes a verification and adjustment should be carried out.

EN

## CHANGING THE BATTERIES

When the LCD screen displays blinking „Battery“ icon, the batteries are used. Open the lid and change the batteries, taking great care with the correct positioning. („+“ on battery to „+“ in battery compartment and „-“ on battery to „-“ in battery compartment).

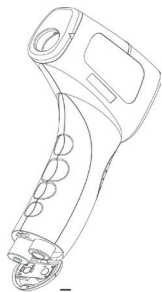A mistake with this could cause damage of the device and compromise the guarantee of your thermometer. Never use rechargeable batteries. **Use only alkaline batteries for single usage.**

## TECHNICAL CHARACTERISTICS

Normal useing condition:

Ambient temperature: 10°C - 40°C (50°F - 104 °F)

Relative humidity: < 85%

Model	JXB 308
Dimensions	138 x 94 x 42 mm
Weight	88 g (without batteries)
Power	DC3 V (2 batteries type AAA, LR03)
Consumption	< 300 mW
Measuring distance	max. 3-5 cm
Automatic power-off	30 sec.

Measuring range:

In Body mode 32°C - 43 °C (89,6 °F - 109,2 °F)

In Surface mode 0 °C - 60 °C (32 °F - 140 °F)

In Room mode 0 °C - 60 °C (32 °F - 140 °F)

Precision:

32°C ~ 34,9°C (89,6 °F ~ 94,8 °F) =  $\pm 0.3^{\circ}\text{C}$  ( $\pm 0,6^{\circ}\text{F}$ )

35°C ~ 42°C (95 °F ~ 107,6 °F) =  $\pm 0.2^{\circ}\text{C}$  ( $\pm 0,4^{\circ}\text{F}$ )

42,1°C ~ 43°C (107,8 °F ~ 109,4°F) =  $\pm 0.3^{\circ}\text{C}$  ( $\pm 0,6^{\circ}\text{F}$ )

According to ASTM Standard E1965-1998 (2009)

### Contents:

1 pc non-contact thermometer Helpmation

2 pcs AAA batteries

1 pc instruction manual

## TROUBLESHOOTING

If you have problems while using your thermometer, please refer to this guide to help resolve the problem. If the problem persists, please contact our authorised service department.

- **The screen displays temperature higher than 42,9 °C (109,2°F):**  
The temperature is in Fahrenheit. Change the measurement to Celsius by pressing °C/°F button.
- **The screen displays temperature lower than 32°C (89,6°F):**  
To take the surface temperature, press the "BODY" button and set to the reading called "Body", If the device is in Surface Temp Mode, the 32°C (89.6°F) temperature displayed is showing the external temperature of your body, rather than the internal.
- **The display shows the message „HI“:**  
When „HI“ is displayed on the thermometer display, the temperature is higher than 43 °C (109.2 °F) in „Body“ mode.
- **The display shows the message „Lo“:**  
When the „Lo“ information appears on the thermometer display, the temperature is lower than the thermometer range. In „Body“ mode, the temperature is below 32 °C (89.6 °F).

### **This may be due to the following:**

- remove hair from the forehead and wipe off any sweat
- the measuring distance is longer than 5 cm from the forehead in BODY mode
- the interval between individual measurements must be 1 minute
- do not be draughty when measuring temperature
- make sure you measure the desired temperature in the correct thermometer setting (BODY, SURFACE, ROOM)

NE

## WARRANTY

Model:

Serial number:

Date of purchase:

Signature and stamp dealer:

The product is guaranteed for 24 months.

The warranty covers defects arising as a result demonstrably defective material, faulty design or bad design.

The warranty does not cover defects caused by transport, normal use of the product (in particular the loss of battery capacity), failure to follow the instructions indicated in the instructions, poor maintenance, mechanical damage, unauthorized intervention in the appliance service. Repairs may only be performed by an authorized service center.

Specifications are based on a condition known at the time of publication. Technical alterations.

This device complies with the EU Directive 93/42/EEC concerning medical products, the ISO 80601-2-56 and the European Standard EN60601-1-2 and is subject to particular precautions with regard to electromagnetic compatibility.

Do not dispose of the product in household waste at the end of its useful life. Please, dispose of this device at your applicable collection point for the recycling of electrical & electronic equipments waste.

Blakar trading s.r.o.  
Czech Republic  
[www.helpmation.com](http://www.helpmation.com)