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Dátum poslednej revízie: 09/2023

medical devices, sterilization by irradiation
www.bioster.cz, www.traumacel.cz

Instruction information. Read carefully! (EN)

Manufacturer: BIOSTER, a.s.
Tejny 621, 664 71 Veverská Bítýška
Czech Republic

RESCUE – Pneumothorax (COLOR YELLOW)

Characteristic

RESCUE – Pneumothorax is a set that is used for the treatment of chest injuries and of traumatic pneumothorax in the open air.

All components are individually packed and each contains the manufacturer's details, product details, batch number, shelf-life, and product use.

Packaging

Absorbent dressing pads 10 x 20 cm, plastic foil 20 x 20 cm, adhesive tape 5 x 500 cm, pair of surgical gloves, sterile.

Application

Injury treatment procedure:

1. After opening the package the sterile surgical gloves should be put on at first.
2. To prevent the bleeding to a chest cavity use a sterile absorbent dressing pad. If blood soaks through, use another piece of dressing.
3. According to the extent of the injury put tightly one or two pieces of the plastic foil over the dressing pads. Fix the plastic foil to a body of an injured person by an adhesive tape, in order to protect the chest against air insufflation.
4. Make sure that the plastic cover fits airtightly. In the opposite case eliminate leakage by putting another piece of tape.

Warnings

The product is intended for single use only. Do not use the product if the packaging is damaged.
Do not take out any objects from the wound.

Storage and shelf life

The product does not require any special storage conditions. Do not use RESCUE – pneumothorax kit after the shelf life date, which is stated on the package. The shelf life date is set in accordance with a component with the earliest shelf life date.

Disposal

All retained biological material will be disposed of as the biological waste.

Date of last revision: 09/2023

medical devices, sterilization by irradiation
www.bioster.cz, www.traumacel.cz

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