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Dátum poslednej revízie: 09/2023

medical devices, sterilization by irradiation
www.bioster.cz, www.traumacel.cz

Instruction information. Read carefully! (EN)

Manufacturer: BIOSTER, a.s.
Tejny 621, 664 71 Veverská Bítýška
Czech Republic

RESCUE - Fractures and bleeding (COLOR BLUE)

Characteristic

RESCUE – Fractures and bleeding is set which serves to treatment of fractures and surface bleeding injuries and wounds. It gives to medic and/or laic suitable fixation and dressing material for treatment of fractures and bleeding injuries perform out of medical facilities, especially for the treatment of scrapes, cuts and lacerations, for giving first aid and to temporarily stop of more serious venous and arterial bleeding, for treatment of dislocations, closed and open fractures, etc.

All components are individually packed and content the information about the manufacturer, product, badge number, date of use, information whether the product is sterile and information for use.

Packaging

Non-sterile gloves, isothermic foil to prevent thermal loss, splints to immobilize dislocations or fractures, bandaging material to fix splints and dressings, triangular bandage for the affected limb, absorbent dressing pads to cover open wounds, bandage to stop arterial bleeding.

Application

Injury treatment procedure:

- wear sterile gloves
- remove the clothes or other subject from the injury area; never remove subjects directly from the wound
- flesh wounds, scrapes or wounds without massive bleeding, cover with sterile absorbent dressing pads and stick to the skin with Omnisilk adhesive plaster (body and head) or with attached bandages (limbs and head)
- if the wound is bleeding or spouting blood, apply a sterile cover in sufficient layers and try to stop bleeding by applying pressure, use elastic tourniquet to maintain the pressure
- if the wound is on a limb, the wound is spouting bright red blood and if it is not possible to stop the bleeding by applying pressure, apply a tourniquet between the body and the wound. Wrap it around a limb and tight it on the second loop until the bleeding stops or is reduced and record the exact time of the tourniquet application
- it is possible to combine both of procedures described above
- if a limb is deformed, attach two Unisplint splints against each other and fix with elastic bandages so that the fracture is immobilized
- in the case of finger injury, attach a finger splint so that it overlaps the fingertip about 1 cm and bandage the finger, palm and wrist
- if an upper limb is affected, create an arm sling from the triangular bandage: tie each end of the two longer sides of the triangular bandage behind the neck and tie a knot on the elbow, then place the limb into the arm sling
- if the skin in the area of injury is damaged or a bone is visible, first treat the open injury according to standard bleeding treatment and then perform immobilization according to standard fracture treatment.
- use isothermic foil to prevent thermal loss or overheating when the injured person is immobilized

Warnings

The product is intended for single use only. Do not use the product if the packaging is damaged.

Storage and shelf life

The product does not require any special storage conditions. Do not use RESCUE – Fractures and bleeding kit after the shelf life date, which is stated on the package. The shelf life date is set in accordance with a component with the earliest shelf life date.

Disposal

All retained biological material will be disposed of as the biological waste.

Date of last revision: 09/2023

medical devices, sterilization by irradiation
www.bioster.cz, www.traumacel.cz

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