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Datum poslednej revízie: 10/2024

medical devices, sterilization by irradiation
www.bioster.cz, www.bioster.eu

Instruction information. Read carefully! (EN)

Manufacturer: BIOSTER, a.s.
Tejny 621, 664 71 Veverská Bítýška
Czech Republic

RESCUE – Thoracic drainage PROFI (COLOR PURPLE-YELLOW)

Characteristic

RESCUE – Thoracic drainage PROFI is a kit designed primarily for use by medical staff to give first aid in the course of a trauma of the lung and subsequent pneumothorax. All components are individually packed and each contains the manufacturer's details, product details, batch number, shelf life, and product use.

All components are individually packed and contain the information about the manufacturer, product, badge number, date of use, information whether the product is sterile and information for use.

Packaging

Sterile scalpel, sterile surgical tweezers with serration, straight sterile scissors cuspate-blunt, sterile needle holder with lock Crile-Wood, sterile haemostatic forceps, sterile drape, Heimlich's valve, catgut non-absorbable knitted non-coated with, adhesive tape, sterile tampon wound from gauze, sterile gloves, sterile compression, PVC drape, collection bag.

Application

Injury treatment procedure:

1. Prepare the aids to perform thoracic drainage – chest tubes, Mesocain %, 1 ampule, syringe 10ml 1pc, green and blue disposable needles and set for thoracic drainage (these aids are not included in the set)
2. Arrange patient's position – if the patient is conscious, half-sitting, with the upper limb behind the head in order to make axillary part accessible. If the patient is unconscious, lay him/her on his/her back with upper limb below his/her head. Uncover his/her chest.
3. The point for inserting the chest tube should be in „safety triangle“ (see the Picture – hatched area) – the border is created from the front edge of upper back muscle, the side edge of chest muscle, lower horizontal line level with the nipple and horizontal line level with armpit.

4. Wear the face mask and sterile gloves.
5. Disinfect the point of puncture (it is not included in the set).
6. Perform a local anaesthesia using 1% Mesocain (it is not included in the set). Perform the anaesthesia in two steps - for anaesthesia of skin, infiltrate the puncture point with blue or orange needle. For the anaesthesia of deeper structures up to pleura, use the green needle. Proceed level with the upper edge of rib to avoid an injury to the neurovascular bundle. Always aspirate before applying the local anaesthetic. An acceptable alternative is also the infiltration of all layers using one green needle.
7. Perform a cut with the attached scalpel, parallel to the rib, a little longer than the diameter of chosen chest tube (usually 1 cm).
8. Dilate the cut. Insert the scissors into the cut so that you gradually dilate it with the blunt part. Remove a mandrin. Please note the chest tube must be inserted without the mandrin!!!

9. Insert a forceps into the cut and penetrate to pleural cavity. Proceed by grasping the closed forceps in the dominant arm and holding its front part with the other hand.
10. Using the forceps, insert the chest tube into the opening. Lead the tube point cranially.
11. Connect the Heimlich valve and collection bag on the end of the chest tube.
12. Fix the inserted chest tube with a stitch (purse-string suture along the outer edge of wound is recommended). Wind the ending of the catgut around the tube and tie with a surgical knot.

Warnings

The product is intended for single use only. Do not use the product if the packaging is damaged.

Storage and shelf life

The storage conditions are stated on the package. Do not use RESCUE – Thoracic drainage PROFI kit after the shelf life date, which is stated on the package. The shelf life date is set in accordance with a component with the earliest shelf life date.

Disposal

All retained biological material will be disposed of as the biological waste.

Date of last revision: 10/2024

medical devices, sterilization by irradiation
www.bioster.cz, www.bioster.eu

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