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Charakteristika

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Dátum poslednej revízie: 09/2023

medical devices, sterilization by irradiation
www.bioster.cz, www.traumacel.cz

PN 011/01a/07

Instruction information. Read carefully! (EN)

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RESCUE - Sudden birth (COLOR GREEN)

Characteristics

RESCUE – Sudden birth kit is intended for doctors and medical staff to give first aid in the course of sudden birth proceeding out of medical facilities, for treatment of a newborn baby and mother, to retain all biological material for later examination and to record vital signs of a baby immediately after birth and the cutting of the umbilical cord and in 5. and 10. minutes periods after cutting of the umbilical cord, according to the standardized evaluation (Apgar score).

All components are individually packed and contain the information about the manufacturer, product, badge number, date of use, information whether the product is sterile and information for use.

Packaging

Pack for mother – non-sterile gloves, sterile gloves, a draw-sheet for women in labour, saline solution and dressing material to wash female genitals, sterile tamponade to stop bleeding from the genitals, sanitary napkins and disposable pants, bag for placenta.

Pack for newborn baby - sterile umbilical clips to ligate the cord, sterile scissors and scalpel to cut the umbilical cord, plaster to fix the sterile dressing on the umbilical cord, textile for the treatment of the newborn, textile and isothermic foil to wrap the newborn, a suction unit to remove the mucus from newborn's mouth and nose, table with marker to record Apgar score.

Application

Baby delivery procedure, treatment of the newborn and mother:

- wear non-sterile gloves
- seat the mother in a comfortable position, with slightly supported chest and head
- wash mother's genitals with saline solution
- underlay a non-sterile draw-sheet
- have the pack for the newborn ready
- help the mother with the childbirth, after the shoulders appear, catch the baby into non-woven textile
- wipe the baby with non-woven textile
- apply two clips on the umbilical cord about 8 cm from newborn's abdominal wall and the third clip apply towards placenta, about 4 cm from previous clips
- wear sterile gloves
- cut the umbilical cord between the clips with scissors or scalpel so that two clips stay on the newborn's umbilical cord and one clip stays at the end towards placenta
- cover the umbilical cord with sterile dressing

- fix the dressing with Omnisilk plaster
- use the suction unit to remove the mucus and the rest of amniotic fluid from the newborn's mouth and nose
- if the newborn does not start breathing and/or crying, start mouth to mouth-and-nose breathing and if there is no show of breathing, continue to resuscitate. If the heart does not start beating again, combine mouth-to-mouth breathing with external cardiac massage in ratio 1 breathe/ 3 chest compressions, with 100 compressions/1 minute frequency. Hold the newborn's chest so that the thumbs are above the breastbone, palms hold the chest and the fingers support the back, creating the support for thumbs' pressure. Continue to resuscitate until the newborn begins to breathe normally or emergency help arrives.
- wrap the newborn to a textile drape and isothermic foil
- finish the placenta delivery
- place the placenta into the attached plastic bag and save for later examination
- if the parturient is bleeding from the genitals, insert tamponade to vagina to reduce or stop the bleeding
- after delivery, provide the mother with sanitary napkins and disposable pants
- fill in the table with Apgar score

Warnings

The product is intended for single use only. Do not use the product if the packaging is damaged.

Storage and shelf life

The product does not require any special storage conditions. Do not use RESCUE – Sudden birth kit after the shelf life date, which is stated on the package. The shelf life date is set in accordance with a component with the earliest shelf life date.

Disposal

All retained biological material will be disposed of as the biological waste.

Date of last revision: 09/2023

medical devices, sterilization by irradiation
www.bioster.cz, www.traumacel.cz

mz1086/7a